

Klarity[®]

AccuCushions[®]

Advanced indexing tools
for highly accurate,
customized support.

Klarity AccuCushions[®] are advanced indexing tools that provide secure, customized head and neck support for patients. These cushions are made from low-density thermoplastic-based materials that become soft and pliable when heated in an oven or water bath. After heating, AccuCushions[®] offer warm and comforting support that has a calming effect on most patients.

Once cool, AccuCushions[®] become firm and rigid, ensuring excellent reproducibility and precise placement for highly accurate treatment. AccuCushions[®] can be used with Silverman headrests, and they also fit under thermoplastic masks to create 360-degree custom support and immobilization.

AccuCushions[®] with extra-fill in the neck can be used without a Silverman headrest, enabling optimal cervical spine support and immobilization.

The inner cushion contains a proprietary mix of high-quality, lightweight polystyrene surrounded by a thin layer of thermoplastic. The soft and stretchable outer covering is made of simple nylon fabric. All AccuCushions[®] are 100% nontoxic, with minimal shrinkage and no odor. AccuCushions[®] are one-patient, but they can be reheated and reformed if needed.



- 100% nontoxic and odorless
- Can be reheated and reshaped
- Increased control of cervical spine area with low attenuation
- Comforting and calming for patients
- Preferred by therapists and clinics worldwide





Head AccuCushions®



RG550-CEL2
 35 x 17.5 cm
 Designed for Elekta's Gamma Knife® Icon™ system, the Icon AccuCushion® should be used with an Icon mask to provide precise fixation and comfortable support.



RG550-H2
 40.5 x 25.8 cm



RG550-M
 20 x 25 cm
 Our standard AccuCushion®. Designed to be used with a Silverman headrest, the RG550-M cushion is our most popular size and is best suited for standard head and neck support.



Head & Shoulder AccuCushion®



RG560-SH2
52 x 47 cm



Body AccuCushion®



RG570-S1
60 x 40 cm



Preparation Guidelines



1. Heat the AccuCushion at 75°C for 10-12 minutes.



2. While the AccuCushion is heating, prepare the patient for setup.




3. When the cushion is ready, place evenly in the desired position.



4. Position the patient on the cushion and gently press the cushion into the desired shape to support the patient.



5. Optional Neoprene strape and velcro can be used to hold the cushion in place while it cools.

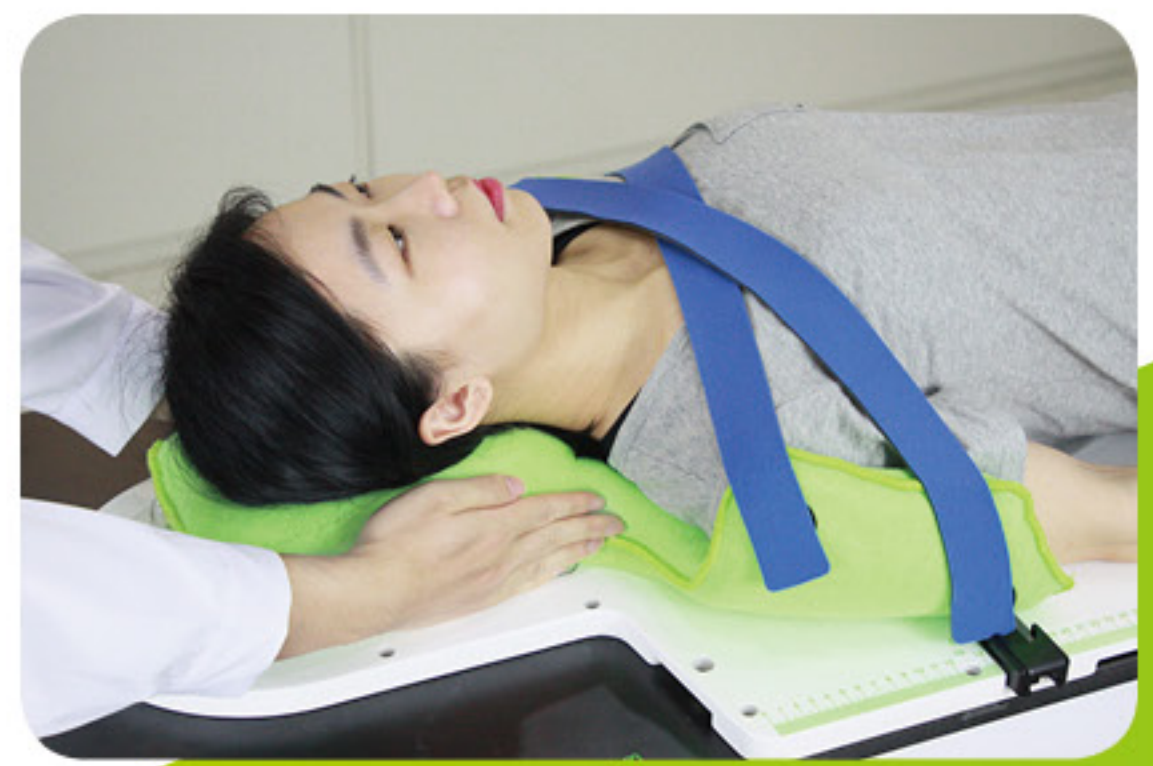

6. Allow the AccuCushion to fully cool and harden for 15 minutes before moving the patient.